

PEDI-AIR ALIGN Instruction Sheet

Strap Assembly



Step 1

Identify Components:
 ~**Shoulder Restraint** (a)
 (Blue 2" wide Strap)
 ~**Chest Strap** (b)
 (Black 3" wide Strap w/ Blue Tabs)
 ~**4 Body Straps** (c)
 (Black 3" wide Straps)
 ~**2 Wrist Restraints** (d)
 (Black 1" wide Straps)
 ~**Pedi~Air~Align** (e)



Step 2

Install **Chest Strap** (black 3" wide strap with blue tabs) in first handhold below Head Drop area.



Step 3

Install **Shoulder Strap** (blue 2" wide strap) through slits in the Head Drop. Secure to exposed Velcro® on Chest Strap.



Step 4

Install **Body Straps** (black 3" wide straps) in the four designated handholds. (Leave center handholds clear for Wrist Restraint attachment.)



Step 5

Loop **Wrist Restraints** (black 1" wide straps) to board through center handholds.

Body Immobilization



Step 1

Always refer to local medical control. Position patient on **Pedi~Air~Align** board with the patient's head in the Head Drop area. **NOTE: Maintain cervical stabilization during immobilization.**



Step 2

Secure **Chest Strap** (black 3" wide strap with blue tabs) over patient. Make sure loop side of Velcro® is exposed.



Step 3


Secure **Shoulder Restraint** (blue 2" wide strap) to exposed Velcro® on Chest Strap in a criss-cross manner.



Step 4

Secure remaining **Body Straps** (black 3" wide straps). Secure **Wrist Restraints** (black 1" wide straps) as needed.

Head Immobilization

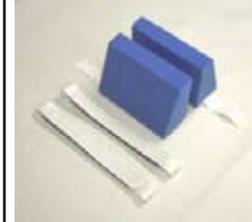

Step 1

Identify Components:
 ~**2 Head Blocks**
 ~**2 Head Straps**
 (White 1" wide straps)

Reorder Part #35843



Step 2

While maintaining cervical stabilization, remove adhesive tab on each **Head Block** and secure to board in desired location.



Step 3

Correct **Head Block** placement shown.



Step 4

Secure one end of each Velcro® **Head Strap** to the corresponding Velcro® attachment area on the back of the **Pedi~Air~Align** board.

Place 1 strap over the patient's forehead and secure to corresponding Velcro® on opposite side. Repeat above process with second strap over patient's chin.



Step 5

Complete Head Immobilization shown.

*Manikin provided by Simulaids, Inc.

NOTE:

Please Refer to Local Medical Direction and Protocol for All Applications of the Iron Duck **Pedi~Air~Align** System and Related Products.



BUILT FOR LIFE 

Sales: 413.377.0030

Order email: OE@IronDuck.com

www.ironduck.com

 facebook.com/ironduckems  [@ironducktweet](https://twitter.com/ironducktweet)